Read Instruction Manual 1 and 2 before use.

This device is a digital monitor operating on the oscillometric method, intended for use in measuring blood pressure and pulse rate of adult patients...

2. Important Safety Information

- 2.1 This warning symbol indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
2.2 Do not use the device in a location with moisture or a location where water may splash on the device...

3. Error messages and troubleshooting

Table with 3 columns: Display/Problem, Cause, Solution. Includes errors like 'START/STOP button pressed while the arm cuff is not applied', 'Air plug is disconnected or not securely connected', etc.

4. Correct Disposal of this Product

This marking shown on the product or its documents indicates that it should not be disposed of with other household waste...

5. Important Information regarding Electro Magnetic Compatibility (EMC)

This device is manufactured by OMRON HEALTHCARE Co., Ltd. and conforms to the EN60601-1-2:2015 Electromagnetic Compatibility (EMC) standard...

6. Warranty

This product is guaranteed by OMRON for a period of 3 years after the date of purchase or starting date of use...

7. General Specifications of Omron Blood Pressure Monitor

Table with 2 columns: Product Category, Specification. Includes categories like Product Category, Display, Measurement method, etc.

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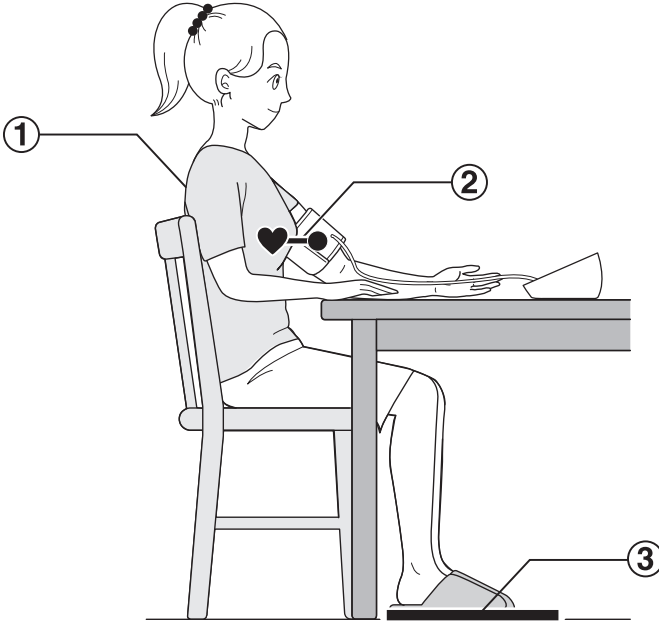
### ① Sit with your back and arm supported.

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### ② Place the arm cuff at the same level as your heart.

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### ③ Keep feet flat, legs uncrossed, remain still and do not talk.

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
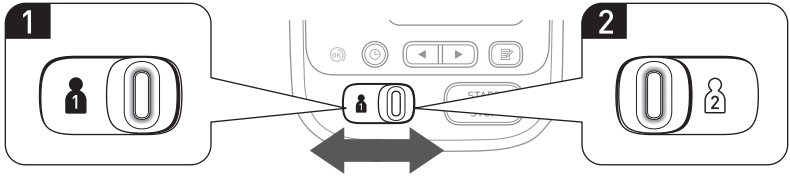
### Switching user ID enables you to save readings for 2 people.

- FR Le changement d'identifiant d'utilisateur vous permet d'enregistrer les résultats de 2 personnes.  
DE Durch Umschalten der Benutzer-ID können Sie Messwerte für 2 Personen speichern.  
IT La modifica dell'ID utente consente di salvare i valori di 2 soggetti.  
ES Cambiar la ID de usuario le permite guardar las lecturas de 2 personas.  
NL Door van gebruikers-ID te veranderen, kunt u metingen van 2 personen opslaan.  
RU Переключение идентификатора пользователя позволяет сохранять показания для 2 людей.  
TR Kullanıcı ID'si değiştirme, 2 kişilik okumaları kaydetmenizi sağlar.  
يمكنك التبديل بين معرفات المستخدم من حفظ قراءات شخصين. AR

## 8 Taking a Measurement

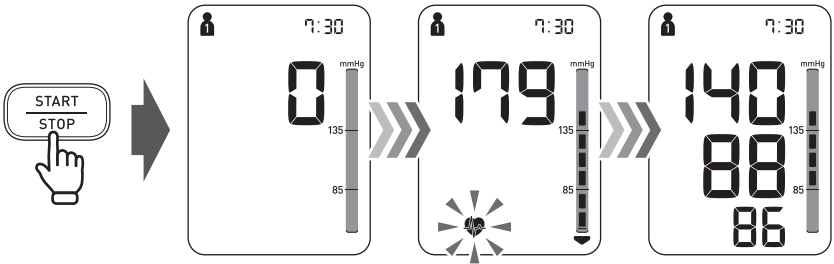- FR Prise d'une mesure NL Een meting uitvoeren  
DE Durchführen einer Messung RU Выполнение измерения  
IT Esecuzione della misurazione TR Ölçüm Yapmak  
ES Tomar una medición (AR) أخذ قياس

### When the START/STOP button is pressed, the measurement is taken and saved automatically.

- FR Lorsque le bouton START/STOP est enfoncé, la mesure est effectuée et enregistrée automatiquement.  
DE Wenn die START/STOP-Taste gedrückt wird, wird die Messung durchgeführt und die Messwerte werden automatisch gespeichert.  
IT Quando si preme il pulsante START/STOP, viene eseguita la misurazione e viene salvata automaticamente.  
ES Al presionar el botón START/STOP se toma la medición y se guarda automáticamente.  
NL Als de START/STOP-knop wordt ingedrukt, wordt de meting uitgevoerd en automatisch opgeslagen.  
RU При нажатии кнопки START/STOP измерение выполняется и сохраняется автоматически.  
TR START/STOP düğmesine basıldığında, ölçüm otomatik olarak alınır ve kaydedilir.  
عند الضغط على زر START/STOP، يأخذ القياس ويحفظ تلقائيًا. AR

## 9 Indication Lights and Symbol Explanations

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**The high blood pressure definition is based on the JNC 7 standard.**

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**Irregular heartbeat symbol:** Appears when an irregular rhythm is detected during a measurement. If it continues to appear, it is recommended to consult your physician. An irregular heart beat rhythm is defined as a rhythm that is 25 % less or 25 % more than the average rhythm detected during a measurement.

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**Movement error symbol :** Appears when your body moves during a measurement. Remove the arm cuff, wait 2-3 minutes and try again.

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E1  
E2  
•  
•

### Error messages or other problems? Refer to:

- FR** Messages d'erreur ou autres problèmes ? Référez-vous à :
- DE** Fehlermeldungen oder sonstige Probleme? Siehe:
- IT** Messaggi di errore o altri problemi? Fare riferimento a:
- ES** ¿Mensajes de error u otros problemas? Consulte:
- NL** Foutmeldingen of andere problemen? Raadpleeg:
- RU** Сообщения об ошибках или другие проблемы? См. следующий документ:
- TR** Hata mesajları veya diğer sorunlar? Şunlara bakın:

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